



Viscount Raised Toilet Seat

VR224C,D,E,F,G,H

VR224CL,DL,FL,GL



Usage and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Viscount Raised Toilet Seat from Aidapt.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Viscount Raised Toilet Seat is available in three sizes - 2",4" and 6" with or without an integral lid, and is designed to fit the majority of UK toilet bowl shapes.

SAFETY

- Consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not sit or step on the lid.
- The Viscount Raised Toilet Seat is intended for indoor use only.
- Carefully check all screws are tightened before use.
- Review weight limit and weight load before use. Do not use exceed the weight limit.
- Do not make any modifications on this device.
- Always check stability before use.
- **Maximum user weight limit of 190kg (30 stones)**

CLEANING

Clean your Viscount Raised Toilet Seat using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure the toilet seat is wiped dry after cleaning.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

INSTALLATION

- 1 – Ensure you inspect the product for any visual damage before use.
- 2 – Carefully remove the existing toilet seat and lid from the toilet.



fig.1

3 – Unscrew the hand wheels of the raised toilet seat (see fig. 1).



fig.2



fig.3

4 – Carefully place the seat onto the toilet bowl (see fig. 2). Position the raised toilet seat correctly on the rim of the toilet ensuring the front lip is outside the bowl. (see fig. 3). To tighten, turn the hand wheels clockwise to tighten.

5 – Check the stability before use.

6 – To remove, simply unscrew the hand wheels and remove the raised toilet seat by lifting up.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com
Email: • sales@aidapt.co.uk