# INSTRUCTIONS FOR USE

## 4-wheeled aluminium rollator with elbow support

Product code: LEXI


### 1. Characteristics

4-wheeled rollator is made of powder coated aluminium frame. It is equipped with brakes with locking mechanism, seat, cane holder and bag. The rollator features height adjustable push handles, elbow supports, and solid 8" wheels.

**The set includes:** rollator frame with seat, two handles with brakes, safety buckle, fixing knob, bag.

### Elements of the rollator: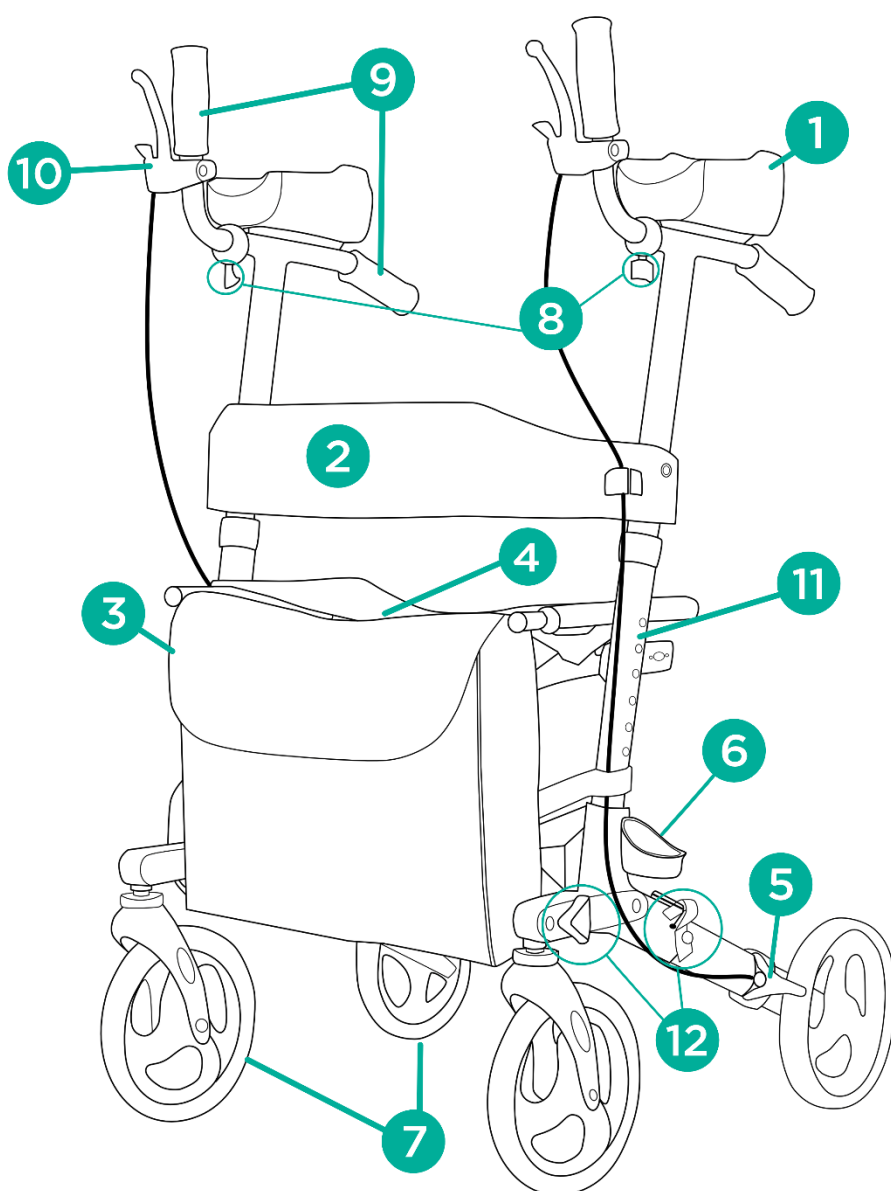

- 1 elbow support
- 2 backrest
- 3 bag
- 4 seat
- 5 brake
- 6 cane holder
- 7 rear and front wheels 8"
- 8 push handle adjustment knob
- 9 push handles
- 10 brake handle with locking mechanism
- 11 height adjustment
- 12 safety buckle and fixing knob


## 2. Application

The rollator is an ancillary equipment and is designed for patients with reduced mobility who need stable support and help while walking. It is especially designed for patients who suffer from rheumatic changes, limited mobility in radiocarpal and elbow joints, and for patients with weakened grip strength. The rollator is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator has a positive effect on the return to activity of the people with locomotion and tilting to erect position problems.


## 3. Assembly

### Folding and unfolding

- 1) Start with unfolding the bottom frame of the rollator.
- 2) Unscrew the fixing knob on the rollator frame and slowly move wheel downwards. Tighten the fixing knob. Repeat on the other side.
- 3) Take out the safety buckle and unlock the blocking pin in mounting openings on the frame (pic 1). Adjust it so both frame parts are aligned horizontally (pic. 2). Secure it with safety buckle and blocking pin (pic. 2).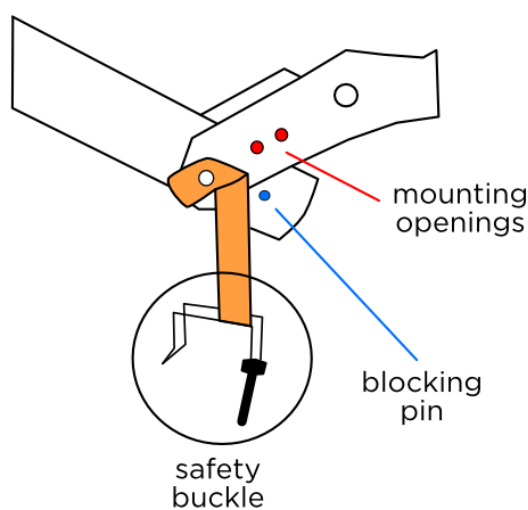

Picture 1

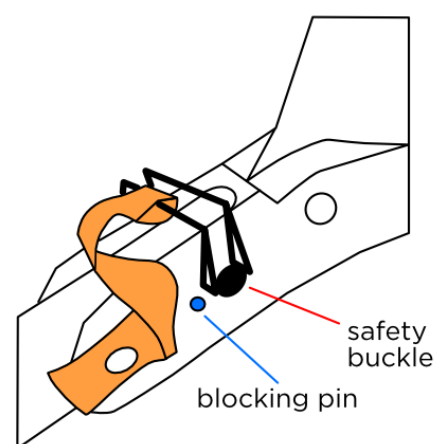Picture 2

- 4) Unfold the rollator by pressing down the seat until you hear a clicking sound.

- 5) Fix the top frame into the bottom frame. Adjust the height with blocking pins. Make sure that height adjustment of both handles is on the same level.
- 6) Attach the bag in front of the rollator. For extra security of the brake cables, attach them to the velcros located on the frame and backrest.
- 7) Rollator is ready to use.

### **Brake function**

- To stop the rollator, pull the brake handle towards the push handle.
- The brakes are equipped with a parking lock function, activated by pulling the handbrakes to a maximum position. To release the lock, grasp the brake handle, squeeze it and pull the locking element (located under the brake handle).

**NOTE!** When using the seat, lock the parking brake.

### **Adjustment of push handles and elbow support**

- To adjust the distance of the push handle, unscrew the adjustment knob which is located under the handle, select appropriate distance and then tighten. Repeat action on the other side. Set both handles at the same length.
- To adjust the height of elbow support, press the blocking pin located on the outer side of the frame, select appropriate height and lock the pin. Set both handles at the same height.

## **4. Technical data**

Overall width	62 cm
Overall depth	66,5 cm
Seat dimensions	46 x 24 cm
Overall height	112-124 cm
Dimensions after folding	74 x 30 x 60 cm
Forearm support height adjustment	98-110 cm
Seat height from the ground	53,5 cm
Weight	8,70 kg

Maximum capacity	136 kg
Frame color	black

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the impurities and check the condition from moving parts of the rollator regularly (hinges, clamping screws, locks).

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.  
Learn more at: **[www.timago.com](http://www.timago.com)**.  
Thank you for choosing Timago!

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