LIGHTMAN START LIGHTMAN START PLUS

04-030-2

04-030-3

04-035-2

04-035-3

LENGVO LYDINIO NEIĞALIOJO VEŽIMĖLIS
VIEGLAIS INVALIDŪDU KRĖSLS
LIGHTWEIGHT WHEELCHAIR
ОБЛЕГЧЁННОЕ ИНВАЛИДНОЕ КРЕСЛО
ODLEHCENÝ INVALIDNÍ VOZÍK

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Att. Nr. 14

PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition. We wish you much success and we hope Kid-Man wheelchair will make a positive impact to your daily living.

INDICATIONS

Wheelchair is designed to transport people with temporary or long-term mobility problems. It can be used both indoors and outdoors. Wheelchair is propelled by moving push rims, mounted on the rear wheels. Ether by attendant.

GENERAL NOTES

- The wheelchairs LightMan Start and LightMan Start Plus are delivered in a cardboard packaging and are ready for use. After unpacking please keep the cardboard packaging if possible. You may need it later for storage or for the return of the wheelchair. Once parcel is received, please check whether it is not damaged during transportation. If you see signs of damage, please make note to transport company on the same time. Please check content of package and if any of below parts are missing, get in touch with your local dealer.
- LightMan wheelchair is available in two different modifications - LightMan Start and LightMan Start Plus. „Plus“ version is upgraded with drum brakes for assistant control. You may identify type of your wheelchair reading a label sticked on the armrest plate and metal cross brace underneath seat. Round sticker with number (for example 39) on skirtguard indicates that size (seat width) of your chair is 39 cm.

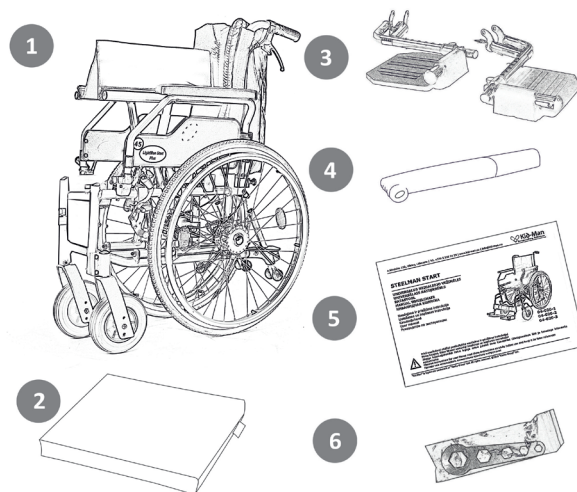
SYMBOLS USED

Attention! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

CONTENT OF PACKAGE

1. 1 x folded wheelchair
2. 1 x cushion
3. 1 x pair of legrest
4. 1 x pump (if rear wheels are inflatable)
5. 1 x user manual
6. 1 x set of adjustment tools

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7 AM/04-035-2/39/16050006

CE

1. Name of product
2. Model
3. Size sticker (seat width)
4. Maximum load
5. Manufacturer
6. Contact details
7. Serial number

PARTS OF THE PRODUCT

1. Backrest
2. Front Castor
3. Armrest
4. Legrest swing lever
5. Legrest
6. Rear self propel wheel
7. Heel strap
8. Brake lever
9. Attendant push handles
10. Adapter block
11. Quick release axle
12. Anti-tipper*
13. Drum brake*
14. Cushion*
15. Velcro straps*

*LightMan Start Plus

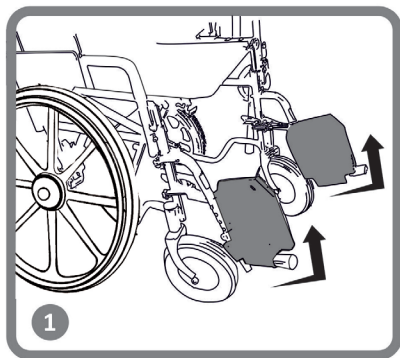
Warning! Completion of your wheelchair can differ a little from a wheelchair described in this manual due to continuous development of the chair.

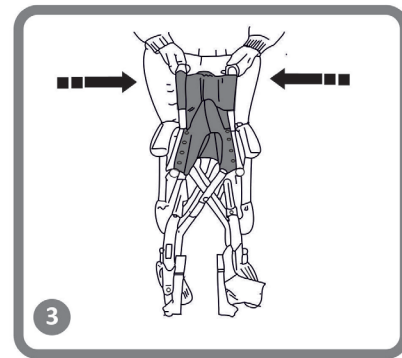
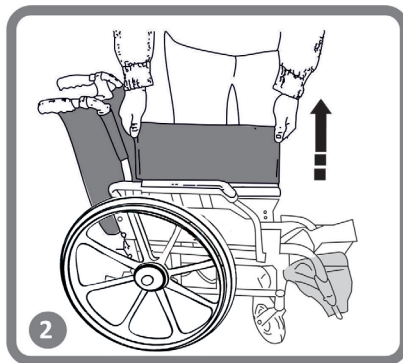
UNFOLDING AND FOLDING THE WHEELCHAIR

To unfold the wheelchair, push with your hands the cross brace apart-down direction until the seat pipes are completely set in the plastic holders. The seat cushion must be completely spread out. To fold wheelchair, first remove legrest or flip up footplates. After, place your hands on the seat cushion at the front and back and pull it up. In order to prepare wheelchair for transportation into car, we suggest to take off the rear wheels gently over the full floating axle. Danger of trapping your finger! Be careful not to grasp the seat pipe.

Pic. No. 1

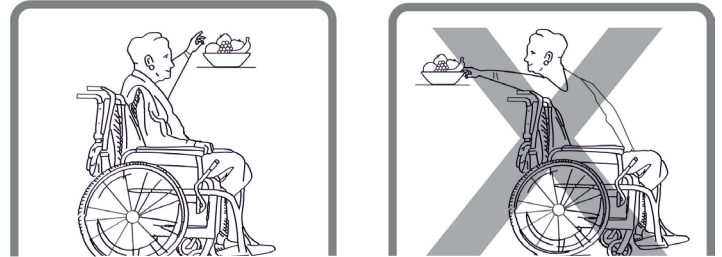
OPERATION

For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

REACHING AN OBJECT FROM THE WHEELCHAIR

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat. (Pic. No.2)

Pic. No. 2

MOVING ONTO THE SIDEWALK

We recommend to have assitant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curbside. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and whenpushes the wheelchair again, until the back wheels go over the curbside. (Pic. No. 3)

Pic. No. 3

TRANSFER

Transfer can be performed alone, assuming you are strong enough. Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place. If there is no support beneath you while moving, please use a board if possible. Fold the legrest up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrest are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.

Pic. No. 4

STAIRS

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order. (Pic. No. 5)

Pic. No.5

ASCENDING/DESCENDING A RAMP

SAFETY WARNINGS

- Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.
- The wheelchair is only suitable for single occupancy.
- Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.
- We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The wheelchair brakes and parking brakes are not suitable for slowing the wheelchair down.
- Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

-  • **Warning!** Do not attempt to lift the wheelchair by any removable parts. Lifting by removable parts of the wheelchair may result in injury to the user or damage to the wheelchair.
-  • **Warning!** Do not attempt to move up or down an incline with a water, ice or oil film.
-  • **Warning!** Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.
-  • **Warning!** Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.
-  • **Warning!** Do not attempt to stop the wheelchair while on a sloped surface.
-  • **Warning!** Repair, cleaning or adjustment of wheelchair should be performed by authorized service person while person is not sitting in a chair.
-  • **Warning!** It is forbidden to exceed the maximum safe load of wheelchair.
-  • **Warning!** The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

HAZARD OF TRAPPING

During assembly, use, handling as well as adjusting the wheelchair, it might happen an entrapment and (or) compression of the part of the user's (accompanying person's body in holes (gaps) between wheelchair's elements. We require to perform any of above mentioned activities with extreme caution. After adjustment of any elements of wheelchair and in order to keep adjusted elements locked in place, please tighten carefully the nuts (bolts).

LIFTING THE WHEELCHAIR

Before carrying the wheelchair, please make sure all removable parts are taken off. The removable parts in the wheelchair can be armrest, legrest, rear wheels and cushions. How to remove them, please refer to this manual. After parts are removed, please to fold the wheelchair. After it is folded, you can easily lift it holding with both hands on the frame. The lifting weight after all parts are removed is around 8-10 kg depending on the size of the chair and model.

TYRES

The front casters of the chair are solid (PU). The rear wheels are pneumatic (04-030-2 series) or solid PU (04-030-3 series). Recommended air pressure is indicated on the tyre. For your safety, please check air pressure in tyres periodically. If you notice air pressure is too low. Please inflate them using the pump which is included into delivery. In case tyres do not keep the air or there is obvious wear of outer tyre, please inform service company immediately. Use of wheelchair with empty air tyres is forbidden!

WARNING OF SMALL PARTS

Please protect children under the age of the five years from easy access to wheelchair. Due to small elements used in the construction, the children can swallow them or choke.

5

CLEANING AND DISINFECTION

Use the moist cloth to clean frame as well as plastic parts. The use of a mild detergent for more stubborn dirt is possible as well. The wheels can be cleaned with a wet synthetic fiber brush (the use of wire brush is not allowed). Seat and back pads can be washed off. Use mild detergent. Use a common non aggressive disinfectant. If you are unsure if the one used by you is suitable for disinfection, please get in touch with local dealer.

THE USER OF CHAIR BY OTHER PERSON

The chair is appropriate for use by others. In order to do this, the wheelchair must first be thoroughly cleaned and disinfected. Then, the all components of the chair must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

DISPOSAL INSTRUCTIONS

The lifetime of wheelchair if used as intended and maintained properly is 5 years. After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste. To have wheelchair properly recycled, please contact your local community, disposal facility or our service department.

TRANSPORT

- To ensure safety and prevent damage while on board, all removable parts (armrests, legrests, rear wheels) must be removed.

CARE AND MAINTENANCE

- Before each use of the wheelchair, the brakes should be checked.
- The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent. Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the cleaners or detergent before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.
- The user/professional assistant should routinely check the chair. It is obligated to replace teared parts (seat and back upholstery, tires).

SPARE PARTS

- Only original spare parts can be used for replacement. Please provide model of wheelchair and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

ARMREST OPERATION

- Wheelchair is equipped with flip up and detachable armrest. (Pic. No. 6) To flip it up, please push plastic lever located in the front of armrest. Keep it pushed and lift armrest gently. We recommend to flip armrest up getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked. In case the need to completely remove armrest occurs, firstly flip it up as explained before. Once armrest is flipped up, find plastic handle in the rear of armrest and pull it out. Once pulled out, armrest is unlocked and you may take it out. To put it back, please repeat actions in opposite direction.

Pic. No. 6

SEAT HEIGHT ADJUSTMENT

- Thanks to front and rear wheels, you are able to adjust seat height. In order to adjust height of steering caster, untighten screw with allen key, pull it out, set caster on desired height and tighten again. (Pic. No. 7) In order to adjust height of rear wheel, untighten 2 screws using spanner and set rear wheel adapter in needed height. After set in place, tighten it again. **Please note, that rear wheel must be removed while making adjustment of adapter.** For safety reasons, we recommend to keep front and rear wheels in the same height. The wheelbase can be extended by turning the adapter by 180°. This increases the stability against tipping backwards.

Pic. No. 7

REAR WHEEL OPERATION

- Your chair has quick release axle, you need no tool to take off the rear wheel. Simply hold frame with one hand and push quick release axle located in the centre of rear wheel. (Pic. No.8) Keep it pushed and pull rear wheel out. To put it back, repeat action in apposite way.

Pic. No. 8

ANTI-TIPPER

- LightMan Start Plus chairs are equipped with anti-tip wheels. They are factory in built, thus requires small adjustments only before use. The aim of anti tippers is to prevent the wheelchair from tipping backwards. They can also be used to help tilt the chair by an accompanying person. The height of the wheels can be adjusted. To do this, pull the silver-coloured disks towards you and slide the tube upwards or downwards along the bracket. If you want to take wheels off, unscrew bolt, take it out and pull device towards.

Pic. No. 9

BRAKING

- Your wheelchair has brakes mounted onto frame from both sides. (Pic. No. 10) Once delivered, brakes are adjusted properly to ensure sufficient braking. In order to apply brakes, please hold braking lever and push it forward. We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes. To do so, please use allen key and untighten two screws located in bottom part of brakes. Then, push it forwards or backwards and tighten screws.

In case you move rear wheel adapter by 180° to extend rear wheel base, it is necessary to re-locate brakes. Please use allen key to untighten brake device, push it backwards and tighten again.

Pic. No. 10

LEGREST OPERATION

The footrests must be installed in the front of chair. Firstly, please insert plastic part into metal tube and push it down. After, move legrest inside until you hear snap. (Pic. No. 11) Once locked in position, legrest stand firmly. Do not mix left and right legrest! After installation, plastic footplate must be faced inwards.

Once leaving wheelchair, legrest must be flipped away or completely removed. Pull plastic lever upwards and move legrest outside. Length of legrest can be adjusted to meet user's needs. Firstly, unscrew bolt using allen key, pull out bolt and tighten it again in needed position.

Pic. No. 11

DRUM BRAKE

Pic. No. 12

Pic. No. 13

- LightMan Start Plus version is equipped with a drum brake, which should only be operated by an accompanying person via the brake lever located on the push handles. (Pic. No. 12). The drum brake should be used as both a locking brake and also as a service brake. (Pic. No. 13) When used as a service brake it provides well-regulated deceleration. The locking brake function is used for parking. To lock the brake, pull the brake lever and push the locking lever upwards. To release the locking brake, pull the brake lever upwards softly and the locking mechanism releases automatically. To achieve optimum braking efficiency, the braking power is set using the adjusting screw. The braking power is increased by unscrewing the adjusting screw and reduced by tightening it.

VELCRO STRAPS

- LightMan Start Plus model has velcro straps on backrest, which help to regulate the level of comfort (Pic. No. 14). In order to regulate the level of comfort, take off the cover of the backrest, unstick velcro straps, set your level of comfort and stick the straps again.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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