## STEELMAN START

**UNIVERSALUS NEIĞALIOJO VEŽIMĖLIS**  
**UNIVERSĀLAIS RATIŅKRĒSLS**  
**RATASTOOL**  
**MANUAL WHEELCHAIR**  
**ИНВАЛИДНАЯ КОЛЯСКА**  
**MECHANICKÝ INVALIDNÍ VOZÍK**

**04-020-1**  
**04-020-2**  
**04-020-3**

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## EN

### PREFACE

• Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition. We wish you much success and we hope Kid-Man wheelchair will make a positive impact to your daily living.

### INDICATIONS

• Intended to be used for persons having temporary or permanent mobility disorders. Suitable to be used both inside and outside.

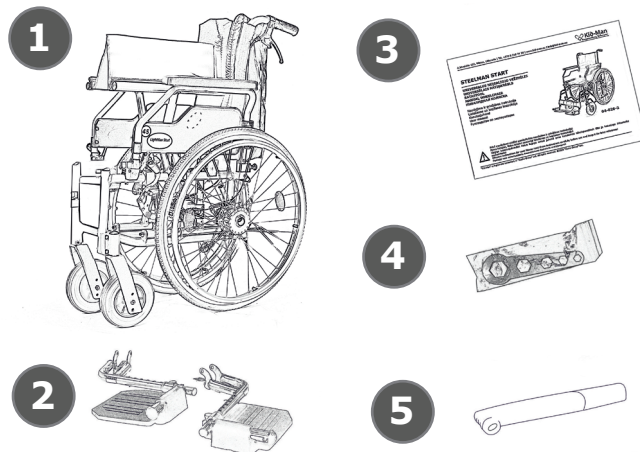
### SYMBOLS USED

• **Attention!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

### CONTENT OF PACKAGE



1. 1 x folded wheelchair
2. 1 x pair of legrest
3. 1 x user manual
4. 1 x set of adjustment tools
5. 1 x pump (if rear wheels are inflatable)

## GENERAL NOTES

38

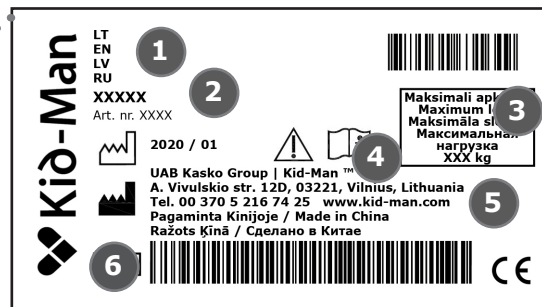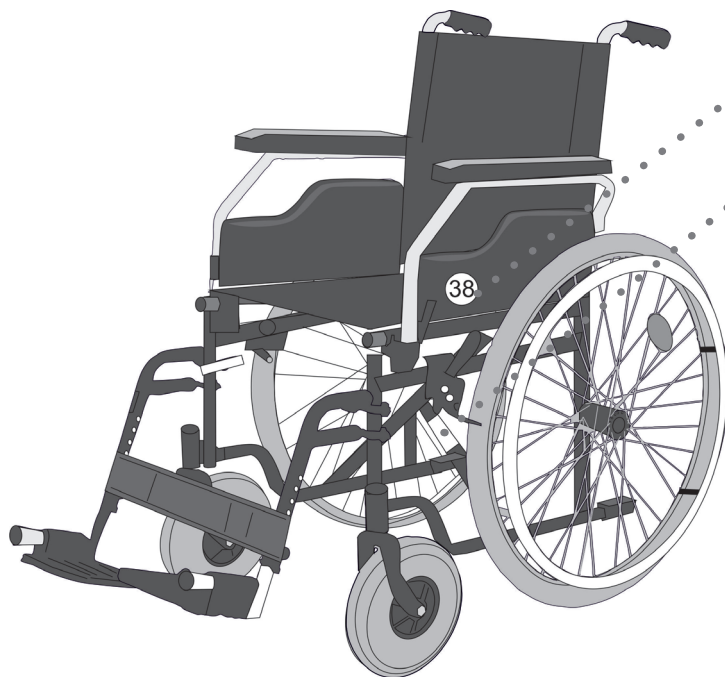
Size sticker  
(seat width)

1. Name of product
2. Model
3. Maximum load
4. Manufacturer
5. Contact details
6. Serial number



## PARTS OF THE PRODUCT



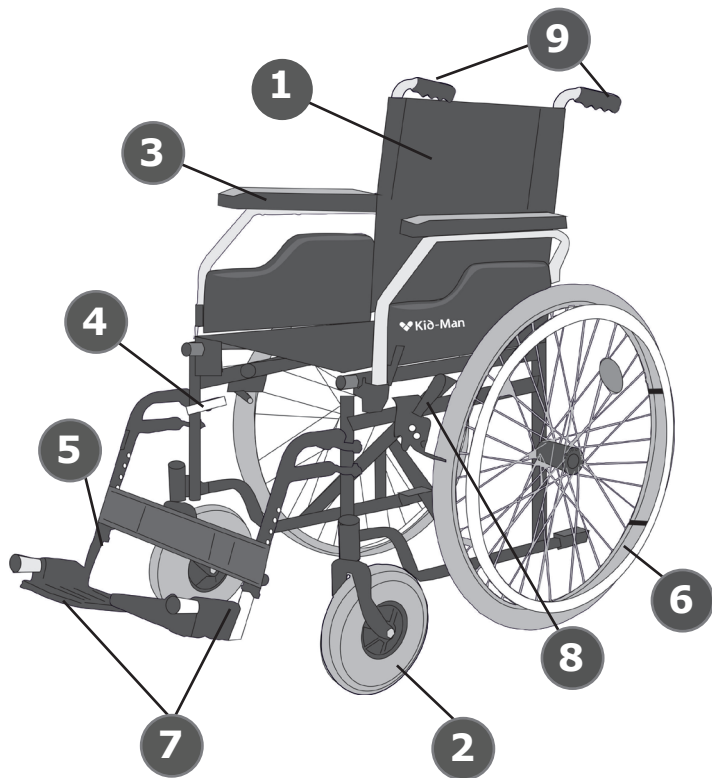
1. Backrest
2. Front Castor
3. Armrest
4. Legrest mounting mechanism
5. Legrest
6. Rear self propel wheel
7. Calf strap
8. Brake lever
9. Attendant push handles

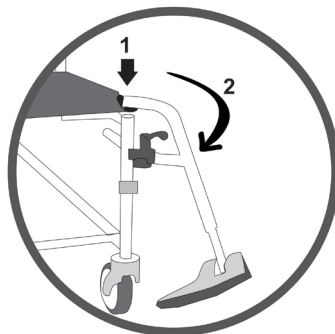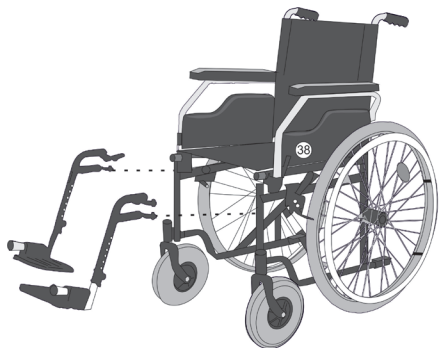**Warning!** Completion of your wheelchair can differ a little from a wheelchair described in this manual due to continuous development of the chair.

**SET UP**

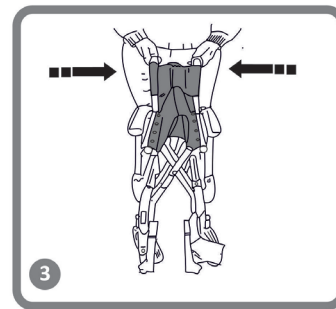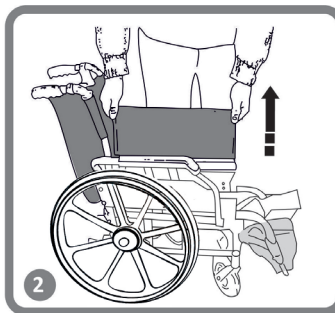
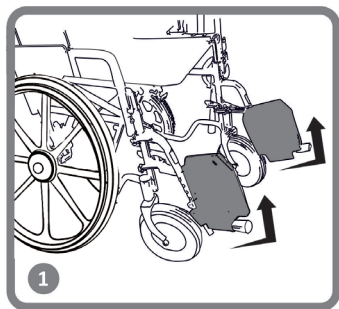
**Warning!** Always ensure the legrest is properly reinstalled and locked in position before use.

**FOLDING THE WHEELCHAIR**

- Firstly, to fold the chair, ensure that the legrests are removed or the footplates flipped up. Once done, commode wheelchair could be folded.

## ARMREST OPERATION

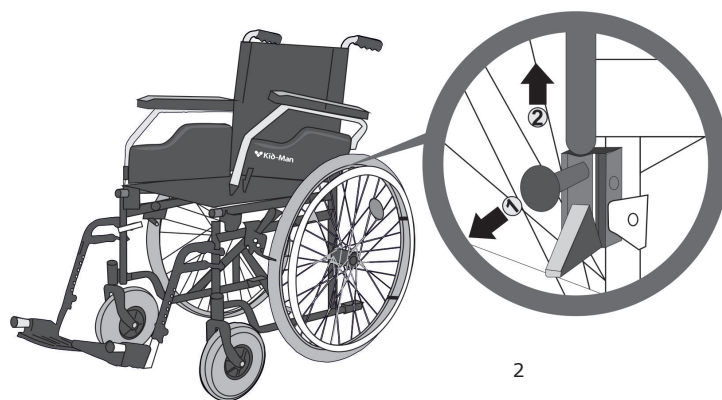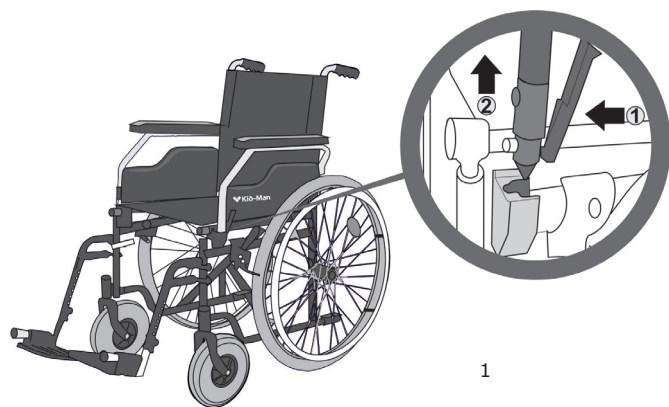• Wheelchair is equipped with flip up and detachable armrest. To flip it up, please push plastic lever located in the front of armrest (2). Keep it pushed and lift armrest gently. We recommend to flip armrest up getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked. In case the need to completely remove armrest occurs, firstly flip it up as explained before. Once armrest is flipped up, find plastic handle in the rear of armrest (3) and pull it out. Once pulled out, armrest is unlocked and you may take it out. To put it back, please repeat actions in opposite direction.


## LEGREST OPERATION

To meet needs of different user, your wheelchair is equipped with height adjustable legrest.


In order to adjust length of legrest, simply push metal button and pull the inned tube in or out depending you want to increase or decrease the length. You may set length in 6 different positions. The length of legrest should be set in position knee angle is bented in around 90°.

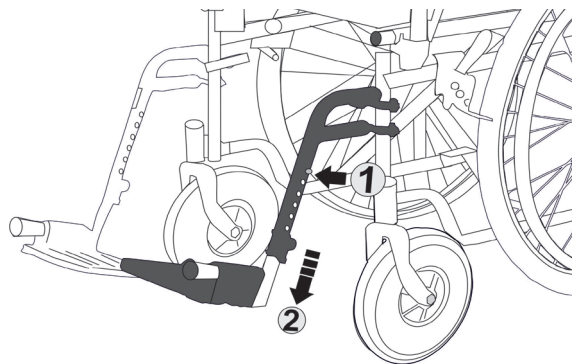There is a risk of changing the point of gravity which leads to bad user's balance as well as risk of back bone deformation and bed sores development if you set length of legrest inproperly.

## REAR WHEEL OPERATION

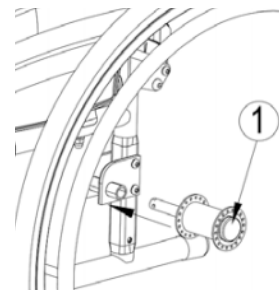- Two different modes of rear wheel fixation are available in 04-020 series. The might be fxed with screw (version 04-020-1) or have quick release axle (1) (versions 04-020-2 and 04-020-3). If you have 04-020-1 version, you need to take a tool located in backrest pocket and loosen screw. Once loosened, hold frame with one hand and pull rear wheel out. To put it back, repeat action in apposite way. If your chair has quick release axle, you need no tool to take off rear wheel. Simply hold frame with one hand and push quick release axle located in the centre of rear wheel. Keep it pushed and pull rear wheel out. To put it back, repeat action in apposite way.


## OPERATION


For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

### REACHING AN OBJECT FROM THE WHEELCHAIR

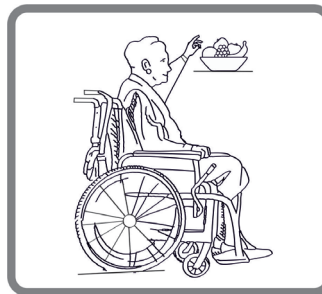Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat. (Pav. Nr. 3)

Pav. Nr. 3

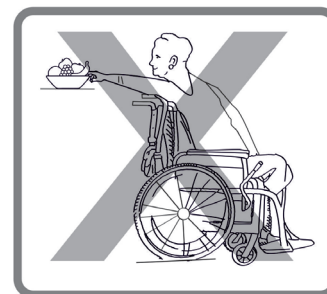
### MOVING ONTO THE SIDEWALK

We recommend to have assistant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curbside. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and when pushes the wheelchair again, until the back wheels go over the curbside. (Pav. Nr. 4)

Pav. Nr. 4
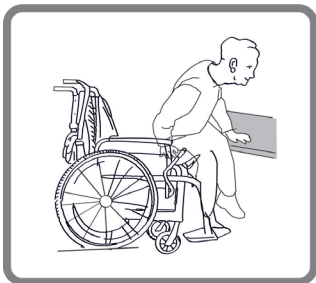
## TRANSFER

Pav. Nr.5


Transfer can be performed alone, assuming you are strong enough. Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place. If there is no support beneath you while moving, please use a board if possible. Fold the legrests up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrests are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.



## STAIRS

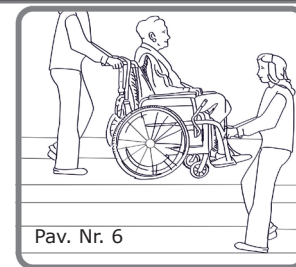We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you have hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order.(Pav. Nr. 6)

Pav. Nr. 6

## ASCENDING/DESCENDING A RAMP
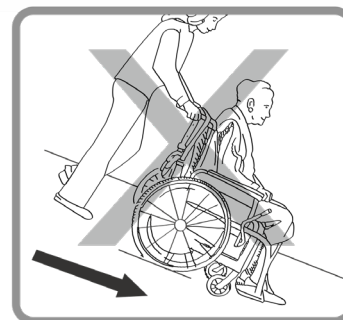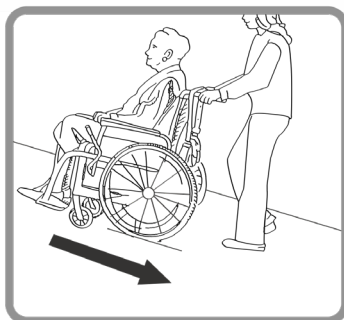
## SAFETY WARNINGS

- Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.
- The wheelchair is only suitable for single occupancy.
- Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.
- We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The wheelchair brakes and parking brakes are not suitable for slowing the wheelchair down.
- Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

- **Warning!** Do not attempt to lift the wheelchair by any removable parts. Lifting by removable parts of the wheelchair may result in injury to the user or damage to the wheelchair.

- **Warning!** Do not attempt to move up or down an incline with a water, ice or oil film.

- **Warning!** Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair

- **Warning!** Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.

- **Warning!** Do not attempt to stop the wheelchair while on a sloped surface.

- **Warning!** All service, cleaning and adjustment should be performed while the wheelchair is unoccupied.

- **Warning!** It is forbidden to exceed the maximum safe load of wheelchair.

- The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

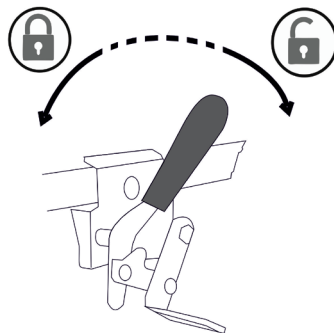
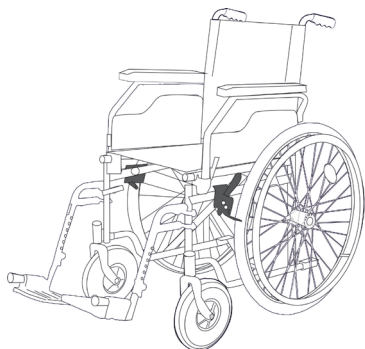
## MOVING

Either the wheelchair user or an attendant can manually propel the wheelchair. If the user is propelling the wheelchair, they should use the handrims attached to the rear wheels. The user can slow the wheelchair by applying pressure to the handrim using their hands. The wheelchair should not be propelled by the user in environments where the user cannot stop the wheelchair promptly and comfortably. If an attendant is propelling the wheelchair, they should push using the push handles.

We do not recommend to use wheelchair in escalators.

## BRAKING

- The wheelchair comes with right and left parking brakes. To apply the brakes, push the handle in the direction shown by the arrow. To unlock the brakes, pull the handle back in the opposite direction.

- Before each use of the wheelchair, the brakes should be checked

## CARE AND MAINTENANCE

- The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent. Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the cleaners or detergent before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.
- The user/professional assistant should routinely check the chair. It is obligated to replace teared parts (seat and back upholstery, tires).

## SPARE PARTS

- Only original spare parts can be used for replacement. Please provide model of wheelchair and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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